

# MediLUME

Please visit our website at:  
[www.medilume.com](http://www.medilume.com)

## Instructions for Refurbishing the Zeiss Pentero Lamp Module

Page 1 of 2

The components:



Standard straight screwdriver



1

Slide red button to left, releasing "Tee" handle on protective cover.




2

Remove protective cover and set aside.



3

Using T20 Torx wrench remove (4) screws on side of housing, remove housing side & baffle.

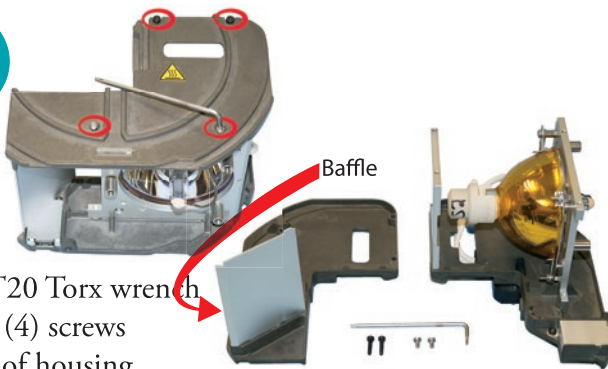


4

Using T20 Torx wrench remove screw on timer housing and set aside



5

Using T25 torx wrench remove screw holding timer PCB

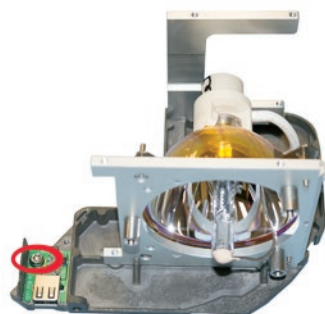


6

Remove expired timer PCB and replace with new. Reinstall timer housing



7

Remove nylon screws from connector housing and remove connector cover.

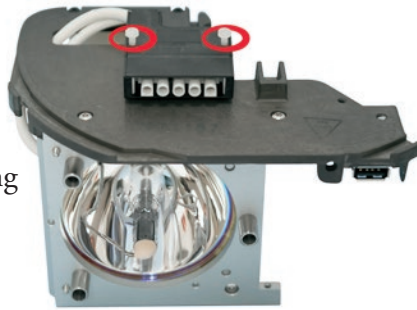

8

Using T20 Torx wrench remove 2 screws on bracket holding lamp. Remove lamp.




9

Install new lamp.




10

Install new lamp connector in housing. Observe connector polarity – as shown.

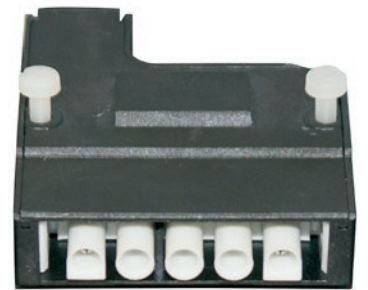


11

Reinstall connector cover.

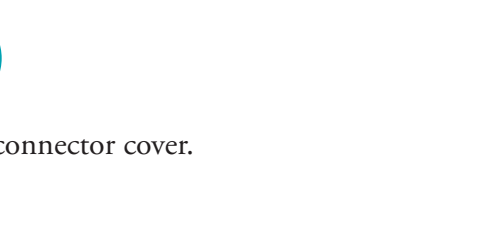


12

Reinstall baffle & side of housing removed in Step 3.

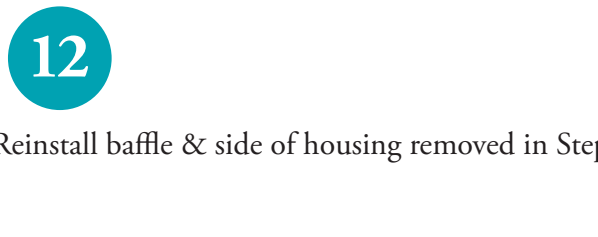

13

Slide protective cover onto housing. Ensure latch is situated as shown.

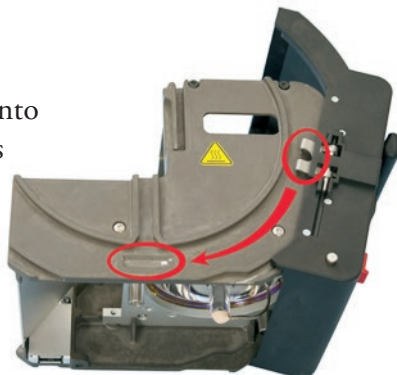

14

Pull “tee” handle up for transport back to microscope.



**Disclaimer:** MediLume is not affiliated with Zeiss and use of any product names or codes are used solely to illustrate physical and technical compatibility of a given lamp or module with the respective equipment.