


Retrofitting The Zeiss Vario 700 Lamp Module



For: Zeiss OPMI Vario 700
(304949-9001-000)

Included in ML-VAR700-R Kit:

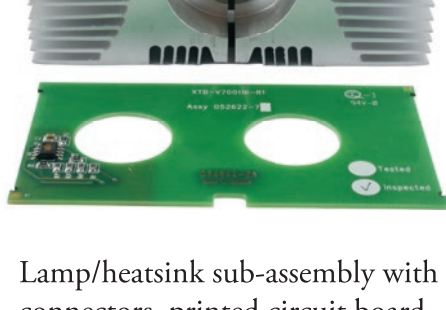

Lamp/heatsink sub-assembly with
connectors, printed circuit board

Tools Required (not included)




- 3/8" Socket driver
- 5/64" Allen key
- Small straight screwdriver

1



Loosen set screws (2)

2

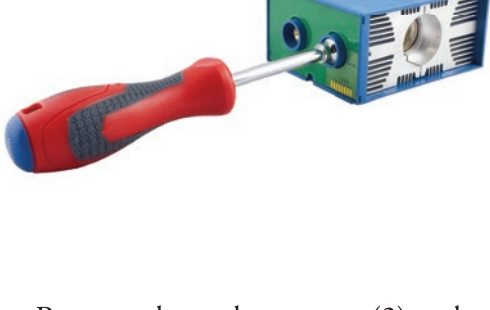

Remove electrode contacts (2) and
remove heatsink/lamp sub-assembly
from housing

3



Insert screwdriver & remove PCB

4



1. Install new PCB, slide in new
heatsink/lamp sub-assembly.
2. Install electrode contacts
3. Tighten set-screws

For more information on our
OEM equivalent lamps and lamp
modules offered,
please visit our website at:
www.medilume.com