

MediLUME

Retrofitting Zeiss Vario 700 & Kinevo 900 Lamp Modules



For: Zeiss OPMI Vario 700
(304949-9001-000)

Zeiss OPMI Kinevo 900
(30949-9002-000)

Included in ML-VAR700-R Kit

ML-304949-9002 Kit:



Lamp/heatsink sub-assembly with connectors, set screws & printed circuit board

Tools Required (not included)



- 3/8" Socket driver
- Small straight screwdriver
- 5/64" Allan key

1




Loosen set screws (2)

2



Remove electrode contacts (2) and slide heatsink/lamp sub-assembly out from housing

3

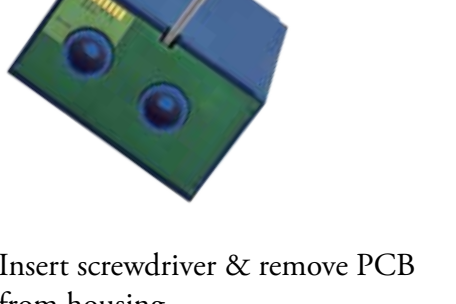

Insert screwdriver & remove PCB from housing.

4



1. Install new PCB, slide in new heatsink/lamp sub-assembly.
2. Install electrode contacts
3. Tighten set-screws

For more information on our OEM equivalent lamps and lamp modules offered, please visit our website at:

www.medilume.com

MediLUME

Disclaimer: MediLume is not affiliated with Zeiss and use of any product names or codes are used solely to illustrate physical and technical compatibility of a given lamp or module with the respective equipment.